



Presentation

The drying cup is an essential accessory to be used with drying capsules.

It ensures a hermetic environment in order to guarantee efficient dehumidification of hearing aids thank to action of dehydrating capsules.

It could be printed or labeled.

Properties

- **Height:** 48mm
- **Diameter:** 66.3mm
- **Cup color:** Transparent (with a label on it)
- **Seal color:** White

The drying cup has a foam disc to prevent damage to the hearing instrument and to keep them stable within the cup during dehumidification.

Lightweight, it is easy to transport.

The drying cup meets the requirement of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety for health.

As manufacturer of Medical Devices, MG Development makes sure of the origin of the raw materials of all its products. And so, all the materials and the coloring agents are complying with the directives and the existing standards (**REACH** for the restriction of chemical substances).

Using advices

Place the capsule, face upwards, under the foam.

Put the hearing aid on the foam with the battery compartment open.

Close the cup and let the capsule act all night long.

Remember to close the cup after each use.

Daily use is recommended, preferably at night.

Regular dehumidification increases the life of hearing systems and preserves their effectiveness.

Precautionary statements

External Use.

Do not swallow.

Keep out of reach of children.

Product composition

Polypropylene

